snoreeze
Snoring relief

nasal strips


snoreeze
Schnarchlinderung

Nasenstrips

How do Snoreeze Nasal Strips work?

Snoreeze Nasal Strips target snoring caused by a cold, allergies or a blocked nose. Their flexible bands gently open the airways and improve airflow to provide effective snoring relief.

Using Snoreeze Nasal Strips is simple.

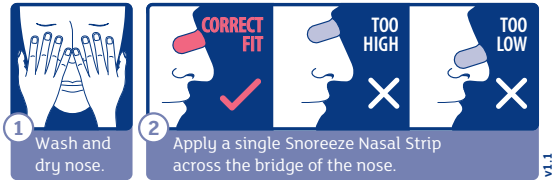Precautions

For external use only. Keep out of reach of small children. Do not place over damaged skin. If skin irritation occurs, discontinue use immediately. Do not use for more than 12 hours per day.

Please note

There is no guaranteed 'cure' for snoring. However, Snoreeze Nasal Strips can relieve the symptoms of snoring to help provide a peaceful night's sleep for you and your partner. Snoreeze Nasal Strips are not a cure for sleep apnoea (a related respiratory medical condition). You may be suffering from sleep apnoea if you stop breathing during sleep, wake-up tired or fall asleep easily during the day. If you experience any of these symptoms it is recommended that you consult your doctor.

For more information visit www.snoreeze.com

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Passion For Life Healthcare (UK) Ltd
Pioneer House, Pioneer Business Park, North Road,
Ellesmere Port, CH65 1AD, United Kingdom
Telephone Helpline 0800 096 11 21

Passion For Life Healthcare (UK) Ltd
Pioneer House, Pioneer Business Park, North Road,
Ellesmere Port, CH65 1AD, Großbritannien
Telefonankunftsstelle +44 (0) 151 550 4700

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